

# PRODUCT INFORMATION

## CARISTA 3



### DESCRIPTION

**Bernafon Carista 3 is a complete family of hearing instruments, suitable for users with mild to severe hearing losses. With Carista 3, your clients will experience high-tech features that bring real benefits, and a hearing instrument that demonstrates exceptional value overall.**

**New functionalities now available in this price category include Bernafon's most advanced Adaptive Noise Reduction Plus and Adaptive Feedback Canceller Plus. Wireless Connectivity is also possible for the first time in a variety of models in the Carista 3 range.**

### AUDIO EFFICIENCY™

#### Speech

- ChannelFree™
- Adaptive Directionality

#### Comfort

- Adaptive Feedback Canceller Plus
- Adaptive Noise Reduction Plus
- Binaural Synchronization

#### Individualization

- Wireless Connectivity
- Language Specific Targets
- REMfit™

### ADDITIONAL FEATURES

#### Technical Features

- Digital signal processing up to 8 kHz
- Multi-Environment Program
- Hydrophobic coating for all BTEs
- Dust and water protection for all BTEs (IP57)

#### Personalization Features

- Data Logging
- Up to 10 listening program options
- 4 freely-assignable program slots
- DAI / FM adapter


# CARISTA 3 BTE PRODUCT OVERVIEW

## COMPACT POWER PLUS BTE


## COMPACT POWER BTE



CA3 CPx

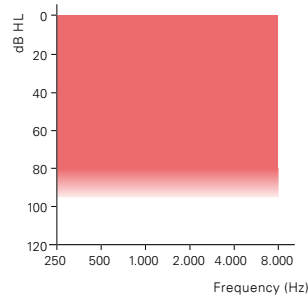

CA3 CP

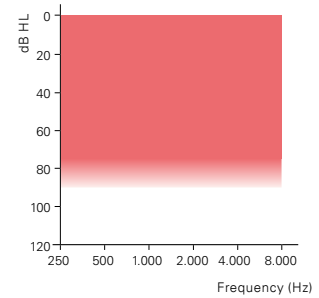

## 2CC COUPLER

## EAR SIMULATOR


		CPx	CP	CPx	CP
OSPL 90, Peak	dB SPL	132*	128	137*	134*
OSPL 90, 1600 Hz	dB SPL	127	122	135*	127
HFA-OSPL 90	dB SPL	123	119	–	–
Full-On Gain, Peak	dB	71	61	77	67
Full-On Gain, 1600 Hz	dB	65	55	73	60
HFA Full-On Gain	dB	59	53	–	–
Reference Test Gain	dB	47	41	60	52
Program Selector		●	●	●	●
Local Volume Control		●	●	●	●
Telecoil		●	●	●	●
FM Adapter		○	○	○	○
DAI Adapter		○	○	○	○
Battery Size		13	13	13	13
Earhook		●	●	●	●
Spira Flex Thin Tube 0.9 / 1.3		○	○	○	○
Microphone System		dual omni	dir	dual omni	dir
Remote Control (RC-P)		○	○	○	○
SoundGate 2 (Bluetooth®)		○	○	○	○
TV Adapter 2		○	○	○	○
Phone Adapter 2		○	○	○	○

● standard ○ optional

"2cc" refers to a coupler according to IEC 60318-5. "Ear simulator" refers to a coupler according to IEC 60318-4. Applied versions: IEC 60118-7:2005, IEC 60118-0:1994 and ANSI S3.22:2009.

\* Special care should be taken when fitting and using a hearing instrument with maximum sound pressure capability in excess of 132 dB SPL (IEC 60318-4) since there may be a risk of impairing the remaining hearing of the hearing instrument user.

NANO BTE NANO RITE M-SPEAKER NANO RITE P-SPEAKER

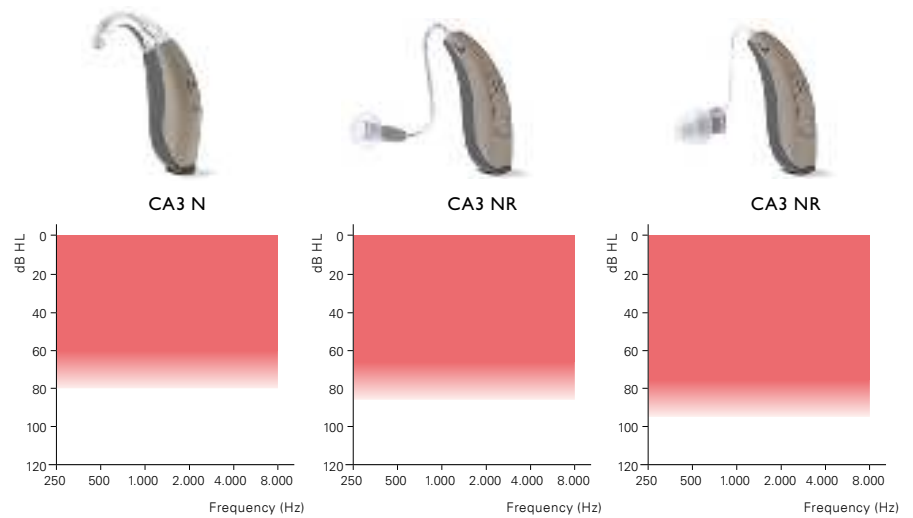

2CC COUPLER EAR SIMULATOR

		2CC COUPLER			EAR SIMULATOR		
		N	NR		N	NR	
			M-SPEAKER	P-SPEAKER		M-SPEAKER	P-SPEAKER
OSPL 90, Peak	dB SPL	122	109	123	128	120	132*
OSPL 90, 1600 Hz	dB SPL	122	105	122	127	114	131
HFA-OSPL 90	dB SPL	115	106	119	–	–	–
Full-On Gain, Peak	dB	49	49	64	55	60	73
Full-On Gain, 1600 Hz	dB	48	44	61	54	52	69
HFA Full-On Gain	dB	42	44	58	–	–	–
Reference Test Gain	dB	36	29	43	47	37	54
Program Selector		●**		●**	●**		●**
Local Volume Control		**		**	**		**
Telecoil		–		●	–		●
FM Adapter		–		–	–		–
DAI Adapter		–		–	–		–
Battery Size		312		312	312		312
Earhook		○		n.a.	○		n.a.
Spira Flex Thin Tube 0.9 / 1.3		●		n.a.	●		n.a.
Microphone System		dir		dir	dir		dir
Remote Control (RC-P)		○		○	○		○
SoundGate 2 (Bluetooth®)		○		○	○		○
TV Adapter 2		○		○	○		○
Phone Adapter 2		○		○	○		○

● standard ○ optional


“2cc” refers to a coupler according to IEC 60318-5. “Ear simulator” refers to a coupler according to IEC 60318-4. Applied versions: IEC 60118-7:2005, IEC 60118-0:1994 and ANSI S3.22:2009.

\* Special care should be taken when fitting and using a hearing instrument with maximum sound pressure capability in excess of 132 dB SPL (IEC 60318-4) since there may be a risk of impairing the remaining hearing of the hearing instrument user.

\*\* Push button can be programmed for volume control use

# CARISTA 3 ITE PRODUCT OVERVIEW

## ITED      ITCPD      ITCD

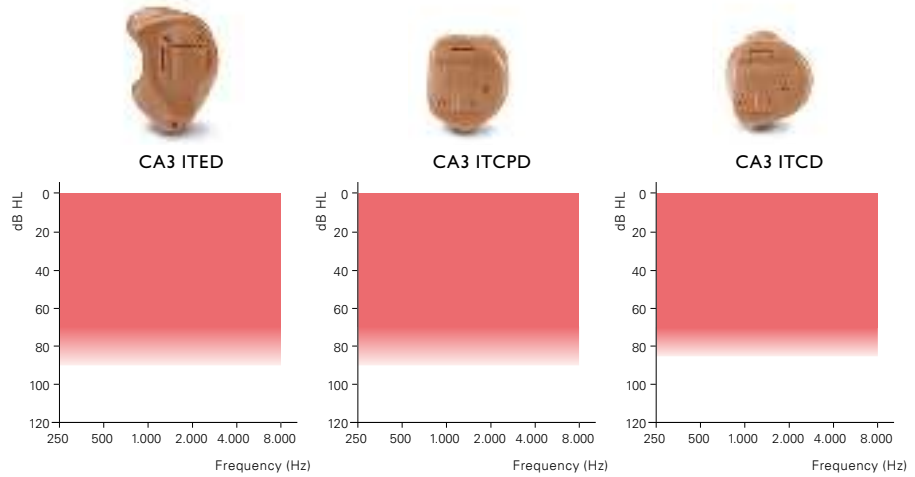


## 2CC COUPLER      EAR SIMULATOR

		ITED	ITCPD	ITCD	ITED	ITCPD	ITCD
OSPL 90, Peak	dB SPL	120	120	118	130	129	128
OSPL 90, 1600 Hz	dB SPL	113	113	111	121	121	119
HFA-OSPL 90	dB SPL	115	114	113	–	–	–
Full-On Gain, Peak	dB	52	53	50	61	62	60
Full-On Gain, 1600 Hz	dB	44	45	39	53	53	48
HFA Full-On Gain	dB	45	46	43	–	–	–
Reference Test Gain	dB	33	33	35	42	43	41
Program Selector		○**	○**	○**	○**	○**	○**
Local Volume Control		**	**	**	**	**	**
Telecoil		○	○	○	○	○	○
Battery Size		13	312	312	13	312	312
Microphone System		dir	dir	dir	dir	dir	dir
Remote Control (RC-P)		○	○	○	○	○	○
SoundGate 2 (Bluetooth®)		○	○	○	○	○	○
TV Adapter 2		○	○	○	○	○	○
Phone Adapter 2		○	○	○	○	○	○

● standard      ○ optional

“2cc” refers to a coupler according to IEC 60318-5. “Ear simulator” refers to a coupler according to IEC 60318-4. Applied versions: IEC 60118-7:2005, IEC 60118-0:1994 and ANSI S3.22:2009.


\*\* Push button can be programmed for volume control use

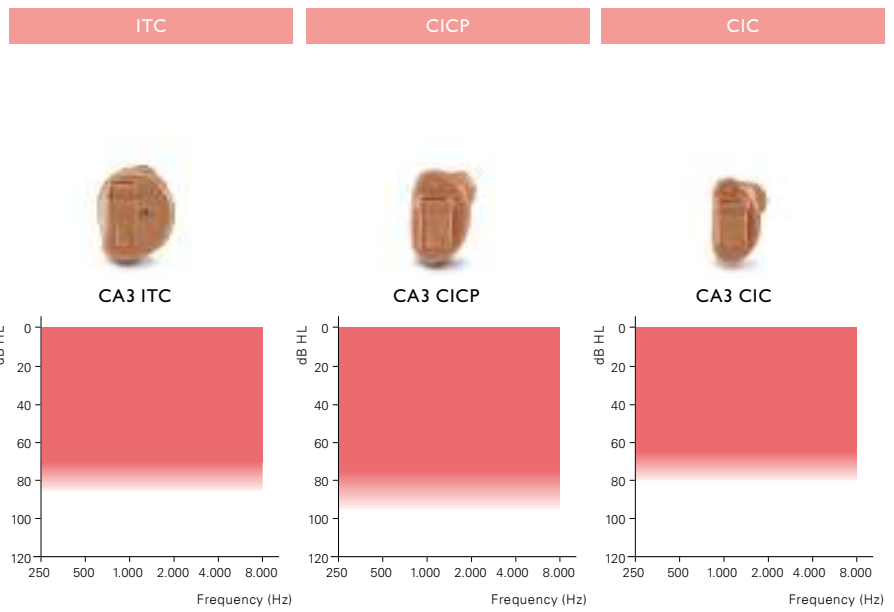

		2CC COUPLER			EAR SIMULATOR		
		ITC	CICP	CIC	ITC	CICP	CIC
OSPL 90, Peak	dB SPL	118	117	108	128	125	118
OSPL 90, 1600 Hz	dB SPL	111	109	100	119	117	108
HFA-OSPL 90	dB SPL	113	112	101	-	-	-
Full-On Gain, Peak	dB	50	48	41	60	57	51
Full-On Gain, 1600 Hz	dB	40	41	31	49	49	40
HFA Full-On Gain	dB	43	43	33	-	-	-
Reference Test Gain	dB	35	34	22	42	42	33
Program Selector		○	○	○	○	○	○
Local Volume Control		○	-	-	○	-	-
Telecoil		○	-	-	○	-	-
Battery Size		312	10	10	312	10	10
Microphone System		omni	omni	omni	omni	omni	omni

“2cc” refers to a coupler according to IEC 60318-5. “Ear simulator” refers to a coupler according to IEC 60318-4.  
 Applied versions: IEC 60118-7:2005, IEC 60118-0:1994 and ANSI S3.22:2009.

● standard ○ optional

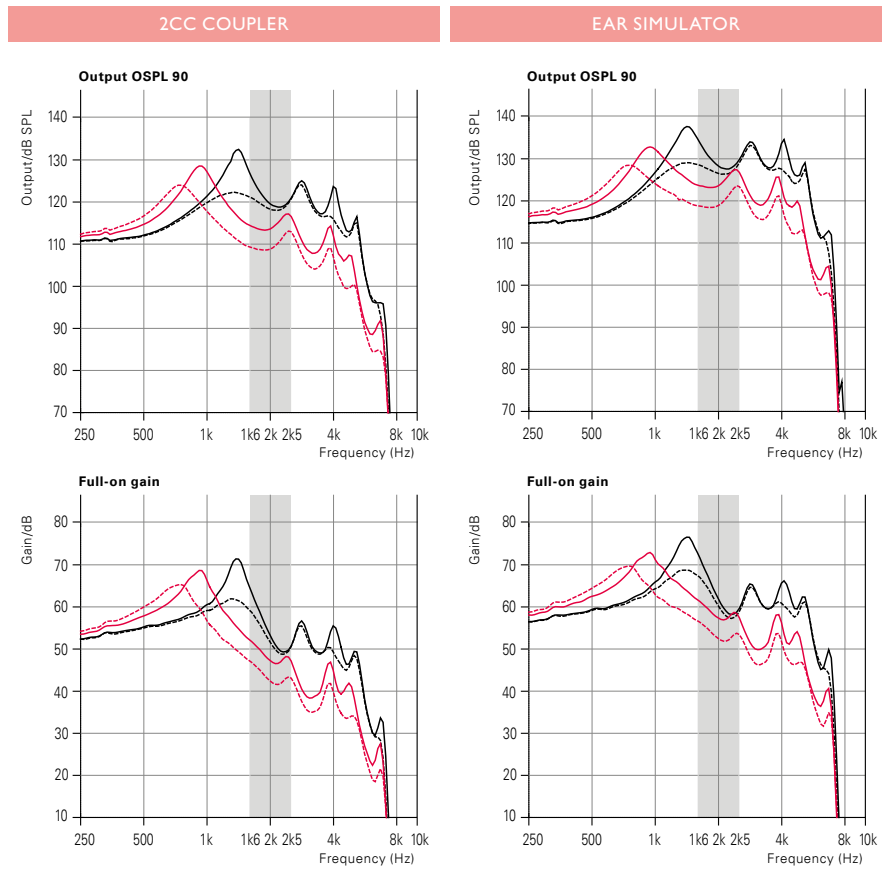



	2CC COUPLER			EAR SIMULATOR		
	EARHOOK	SPIRA FLEX 1.3	SPIRA FLEX 0.9	EARHOOK	SPIRA FLEX 1.3	SPIRA FLEX 0.9
OSPL 90, Peak	132*	128	124	137*	133*	128
OSPL 90, 1600 Hz	127	114	109	135*	124	119
HFA-OSPL 90	123	119	113	-	-	-
Full-On Gain, Peak	71	69	65	77	73	69
Full-On Gain, 1600 Hz	65	52	47	73	62	57
HFA Full-On Gain	59	55	49	-	-	-
Reference Test Gain	47	44	38	60	49	44
Quiescent Current	1.1	1.1	1.1	1.1	1.1	1.1
Operating Current	1.6	1.6	1.6	1.2	1.2	1.2
Battery Size	13			13		
Distortion 500/800/1600 Hz	<5/<4/<2	<4/<2/<2	<2/<2/<2	<6/<5/<2	<4/<2/<2	<2/<2/<2
Frequency Range	100-5600	100-5200	100-5500	-	-	-
Equivalent Input Noise <sup>1)</sup>	21	19	22	14	19	20
Telecoil 1 mA/m 1600 Hz, IEC	93	80	74	102	89	84
Telecoil HFA SPLITS	100	95	90	-	-	-

<sup>1)</sup> Technical data measured with expansion, corresponding to the test box measurement settings.

“2cc” refers to a coupler according to IEC 60318-5. “Ear simulator” refers to a coupler according to IEC 60318-4.

Applied versions: IEC 60118-7:2005, IEC 60118-0:1994 and ANSI S3.22:2009.

\* Special care should be taken when fitting and using a hearing instrument with maximum sound pressure capability in excess of 132 dB SPL (IEC 60318-4) since there may be a risk of impairing the remaining hearing of the hearing instrument user.

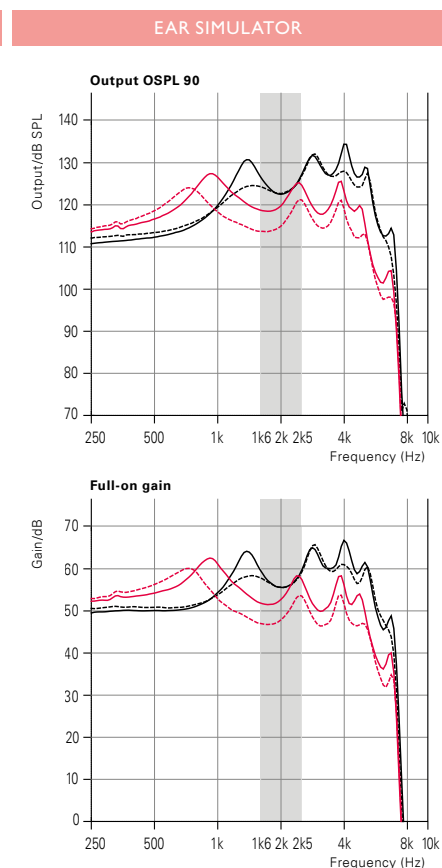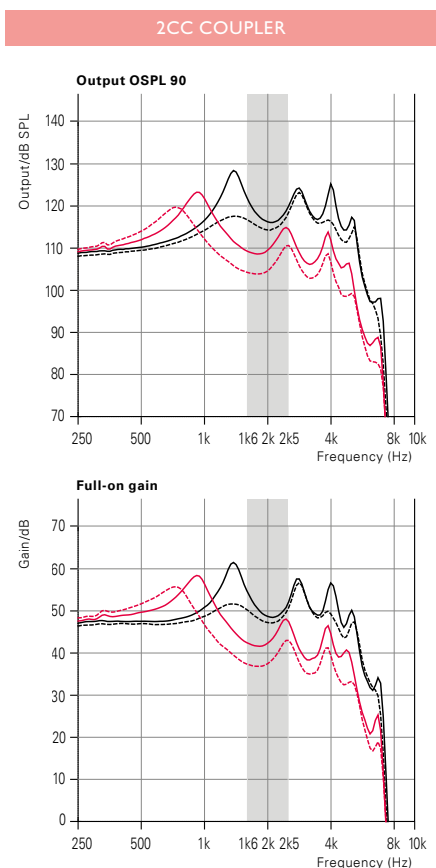

**2CC COUPLER**

	EARHOOK	SPIRA FLEX 1.3	SPIRA FLEX 0.9
OSPL 90, Peak	128	123	120
OSPL 90, 1600 Hz	122	109	104
HFA-OSPL 90	119	115	109
Full-On Gain, Peak	61	58	56
Full-On Gain, 1600 Hz	55	42	37
HFA Full-On Gain	53	49	42
Reference Test Gain	41	37	31
Quiescent Current	1.1	1.1	1.1
Operating Current	1.2	1.2	1.2
Battery Size	13		
Distortion 500/800/1600 Hz	<2/<2/<2	<2/<2/<2	<2/<2/<2
Frequency Range	100-6100	100-5500	100-5800
Equivalent Input Noise <sup>1)</sup>	21	17	20
Telecoil 1 mA/m 1600 Hz, IEC	83	70	65
Telecoil HFA SPLITS	90	92	87

**EAR SIMULATOR**


	EARHOOK	SPIRA FLEX 1.3	SPIRA FLEX 0.9
OSPL 90, Peak	134*	127	124
OSPL 90, 1600 Hz	127	119	114
HFA-OSPL 90	–	–	–
Full-On Gain, Peak	67	62	60
Full-On Gain, 1600 Hz	60	52	47
HFA Full-On Gain	–	–	–
Reference Test Gain	52	44	39
Quiescent Current	1.1	1.1	1.1
Operating Current	1.2	1.2	1.2
Battery Size	13		
Distortion 500/800/1600 Hz	<4/<2/<2	<2/<2/<2	<2/<2/<2
Frequency Range	–	–	–
Equivalent Input Noise <sup>1)</sup>	20	22	23
Telecoil 1 mA/m 1600 Hz, IEC	92	80	75
Telecoil HFA SPLITS	–	–	–

OSPL 90, Peak	dB SPL
OSPL 90, 1600 Hz	dB SPL
HFA-OSPL 90	dB SPL
Full-On Gain, Peak	dB
Full-On Gain, 1600 Hz	dB
HFA Full-On Gain	dB
Reference Test Gain	dB
Quiescent Current	mA
Operating Current	mA
Battery Size	
Distortion 500/800/1600 Hz	%
Frequency Range	Hz
Equivalent Input Noise <sup>1)</sup>	dB(A)
Telecoil 1 mA/m 1600 Hz, IEC	dB SPL
Telecoil HFA SPLITS	dB SPL

<sup>1)</sup> Technical data measured with expansion, corresponding to the test box measurement settings.

“2cc” refers to a coupler according to IEC 60318-5. “Ear simulator” refers to a coupler according to IEC 60318-4. Applied versions: IEC 60118-7:2005, IEC 60118-0:1994 and ANSI S3.22:2009.



\* Special care should be taken when fitting and using a hearing instrument with maximum sound pressure capability in excess of 132 dB SPL (IEC 60318-4) since there may be a risk of impairing the remaining hearing of the hearing instrument user.

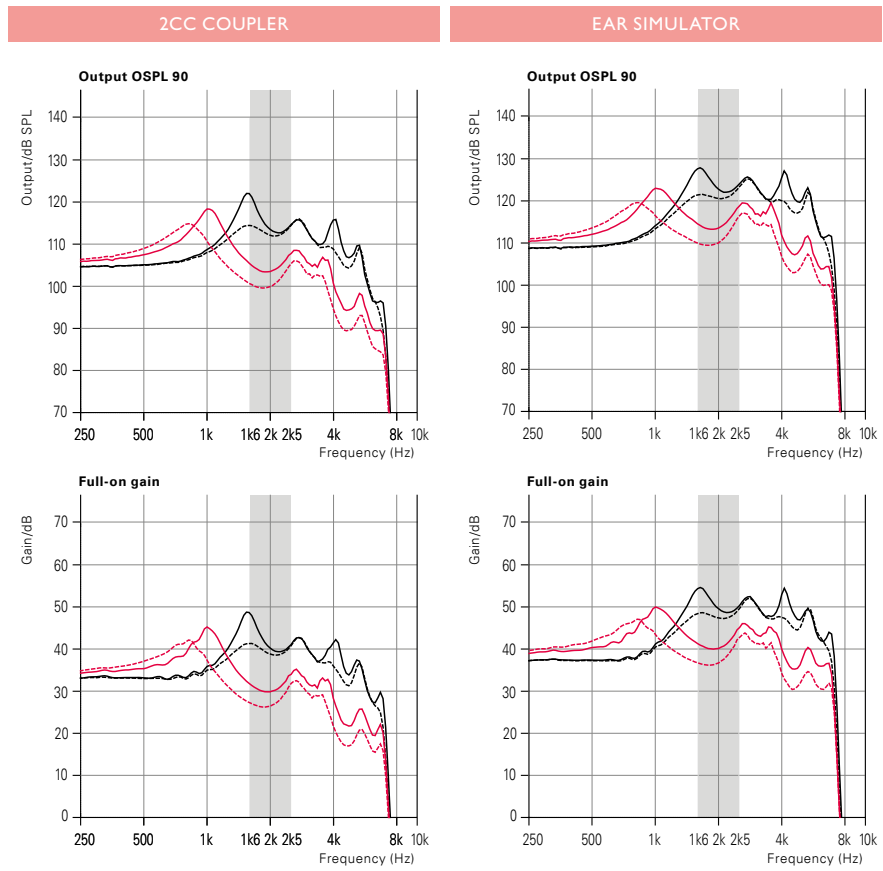

	2CC COUPLER			EAR SIMULATOR		
	EARHOOK	SPIRA FLEX 1.3	SPIRA FLEX 0.9	EARHOOK	SPIRA FLEX 1.3	SPIRA FLEX 0.9
OSPL 90, Peak	122	118	115	128	123	119
OSPL 90, 1600 Hz	122	105	101	127	114	110
HFA-OSPL 90	115	110	105	-	-	-
Full-On Gain, Peak	49	46	42	55	50	47
Full-On Gain, 1600 Hz	48	32	27	54	41	36
HFA Full-On Gain	42	37	32	-	-	-
Reference Test Gain	36	31	26	47	34	30
Quiescent Current	1.1	1.1	1.1	1.1	1.1	1.1
Operating Current	1.1	1.1	1.1	1.1	1.1	1.1
Battery Size	312			312		
Distortion 500/800/1600 Hz	<2/<2/<2	<2/<2/<2	<2/<2/<2	<3/<2/<2	<2/<2/<2	<2/<2/<2
Frequency Range	100-7100	100-7100	100-7100	-	-	-
Equivalent Input Noise <sup>1)</sup>	15	14	15	12	18	20

<sup>1)</sup> Technical data measured with expansion, corresponding to the test box measurement settings. "2cc" refers to a coupler according to IEC 60318-5. "Ear simulator" refers to a coupler according to IEC 60318-4. Applied versions: IEC 60118-7:2005, IEC 60118-0:1994 and ANSI S3.22:2009.



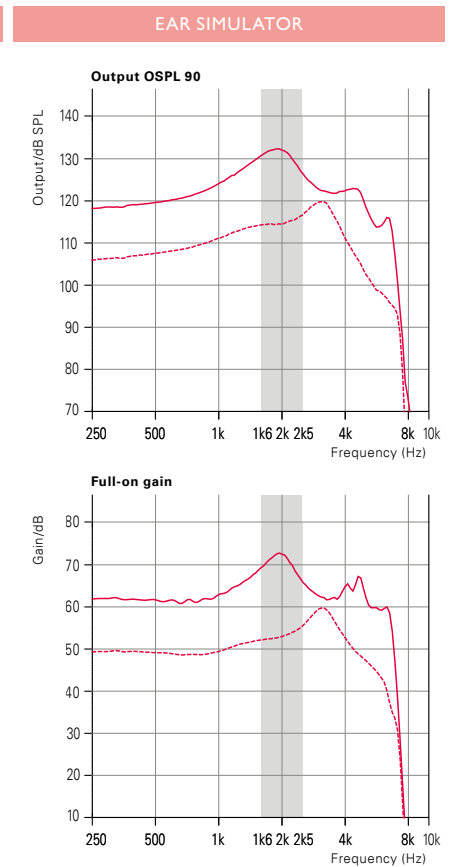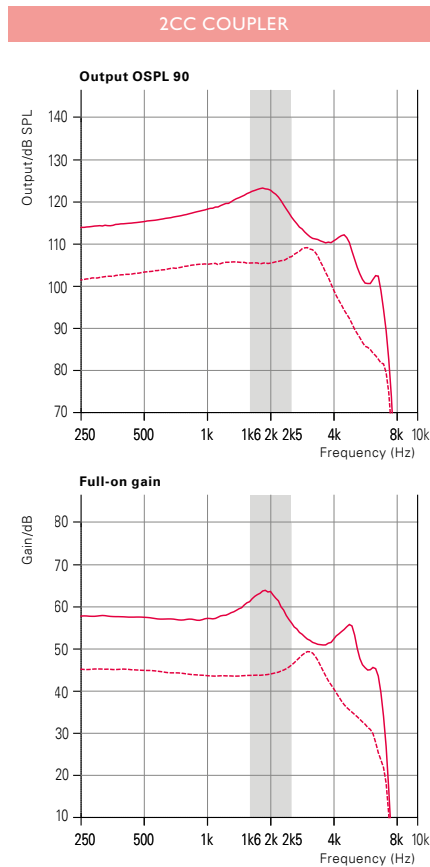

2CC COUPLER

	M-SPEAKER	P-SPEAKER
OSPL 90, Peak	109	123
OSPL 90, 1600 Hz	105	122
HFA-OSPL 90	106	119
Full-On Gain, Peak	49	64
Full-On Gain, 1600 Hz	44	61
HFA Full-On Gain	44	58
Reference Test Gain	29	43
Quiescent Current	1.1	1.1
Operating Current	1.1	1.4
Battery Size	312	
Distortion 500/800/1600 Hz	<2/<2/<2	<2/<2/<2
Frequency Range	100-6700	100-6900
Equivalent Input Noise <sup>1)</sup>	17	16
Telecoil 1 mA/m 1600 Hz, IEC	70	88
Telecoil HFA SPLITS	74	89

EAR SIMULATOR

	M-SPEAKER	P-SPEAKER
OSPL 90, Peak	120	132*
OSPL 90, 1600 Hz	114	131
HFA-OSPL 90	-	-
Full-On Gain, Peak	60	73
Full-On Gain, 1600 Hz	52	69
HFA Full-On Gain	-	-
Reference Test Gain	37	54
Quiescent Current	1.1	1.1
Operating Current	1.1	1.2
Battery Size	312	
Distortion 500/800/1600 Hz	<3/<3/<2	<2/<2/<2
Frequency Range	-	-
Equivalent Input Noise <sup>1)</sup>	19	14
Telecoil 1 mA/m 1600 Hz, IEC	79	95
Telecoil HFA SPLITS	-	-

<sup>1)</sup> Technical data measured with expansion, corresponding to the test box measurement settings.

“2cc” refers to a coupler according to IEC 60318-5. “Ear simulator” refers to a coupler according to IEC 60318-4. Applied versions: IEC 60118-7:2005, IEC 60118-0:1994 and ANSI S3.22:2009.

\* Special care should be taken when fitting and using a hearing instrument with maximum sound pressure capability in excess of 132 dB SPL (IEC 60318-4) since there may be a risk of impairing the remaining hearing of the hearing instrument user.



CA3 ITED





CA3 ITCPD





CA3 ITCD

— ITED  
 - - - ITCPD  
 — ITCD

2CC COUPLER

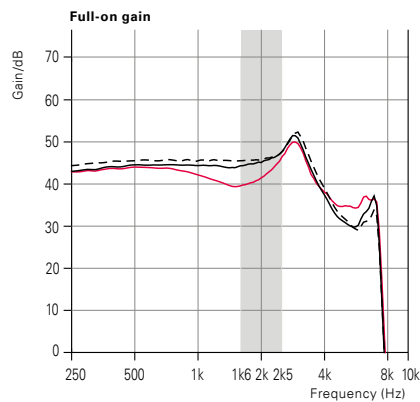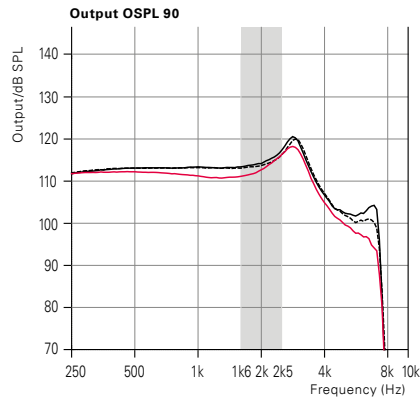

EAR SIMULATOR

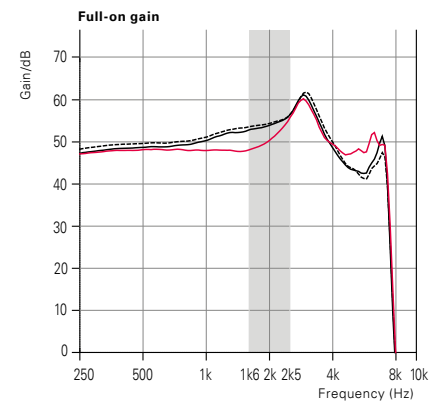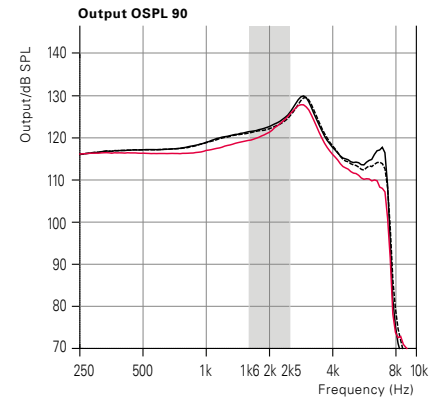

2CC COUPLER



EAR SIMULATOR

		ITED	ITCPD	ITCD	ITED	ITCPD	ITCD
OSPL 90, Peak	dB SPL	120	120	118	130	129	128
OSPL 90, 1600 Hz	dB SPL	113	113	111	121	121	119
HFA-OSPL 90	dB SPL	115	114	113	-	-	-
Full-On Gain, Peak	dB	52	53	50	61	62	60
Full-On Gain, 1600 Hz	dB	44	45	39	53	53	48
HFA Full-On Gain	dB	45	46	43	-	-	-
Reference Test Gain	dB	33	33	35	42	43	41
Quiescent Current	mA	1.1	1.1	1.1	1.1	1.1	1.1
Operating Current	mA	1.1	1.1	1.2	1.1	1.1	1.1
Battery Size		13	312	312	13	312	312
Distortion 500/800/1600 Hz	%	<2/<2/<2	<2/<2/<2	<2/<2/<2	<2/<2/<2	<2/<2/<2	<2/<2/<2
Frequency Range	Hz	100-7500	100-7300	100-7500	-	-	-
Equivalent Input Noise <sup>1)</sup>	dB(A)	19	19	20	20	20	23
Telecoil 1 mA/m 1600 Hz, IEC	dB SPL	77	77	71	85	84	80
Telecoil HFA SPLITS	dB SPL	92	92	91	-	-	-



<sup>1)</sup> Technical data measured with expansion, corresponding to the test box measurement settings.  
 "2cc" refers to a coupler according to IEC 60318-5. "Ear simulator" refers to a coupler according to IEC 60318-4.  
 Applied versions: IEC 60118-7:2005, IEC 60118-0:1994 and ANSI S3.22:2009.



2CC COUPLER

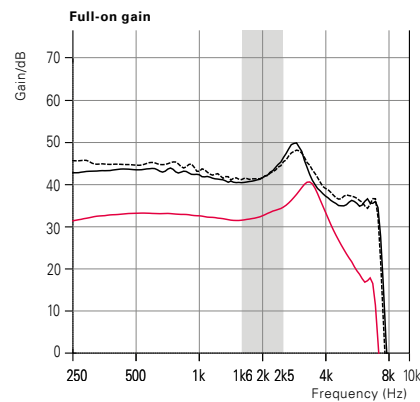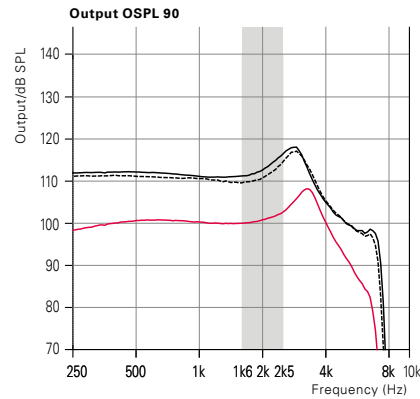

EAR SIMULATOR

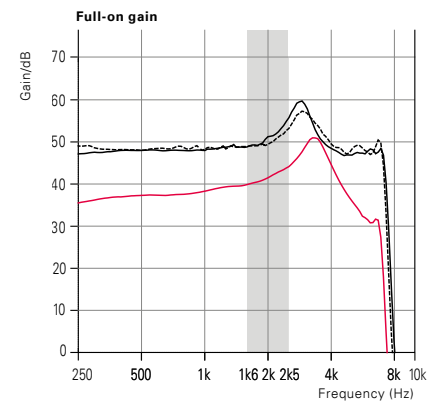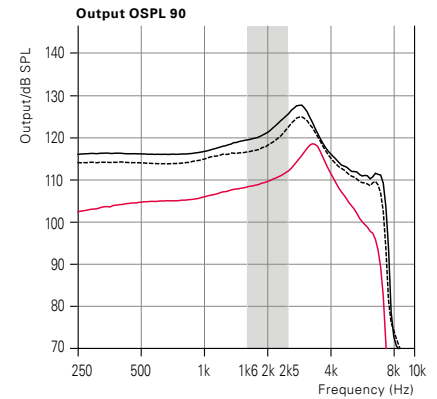

2CC COUPLER

EAR SIMULATOR

		ITC	CICP	CIC	ITC	CICP	CIC
OSPL 90, Peak	dB SPL	118	117	108	128	125	118
OSPL 90, 1600 Hz	dB SPL	111	109	100	119	117	108
HFA-OSPL 90	dB SPL	113	112	101	-	-	-
Full-On Gain, Peak	dB	50	48	41	60	57	51
Full-On Gain, 1600 Hz	dB	40	41	31	49	49	40
HFA Full-On Gain	dB	43	43	33	-	-	-
Reference Test Gain	dB	35	34	22	42	42	33
Quiescent Current	mA	0.8	0.8	0.7	0.8	0.8	0.7
Operating Current	mA	0.9	1	0.8	0.8	0.8	0.8
Battery Size		312	10	10	312	10	10
Distortion 500/800/1600 Hz	%	<2/<2/<2	<2/<2/<2	<2/<2/<2	<2/<2/<2	<2/<2/<2	<2/<2/<2
Frequency Range	Hz	100-7500	100-7300	100-6900	-	-	-
Equivalent Input Noise <sup>1)</sup>	dB(A)	21	21	22	25	23	24
Telecoil 1 mA/m 1600 Hz, IEC	dB SPL	72	-	-	80	-	-
Telecoil HFA SPLITS	dB SPL	91	-	-	-	-	-

<sup>1)</sup> Technical data measured with expansion, corresponding to the test box measurement settings.

“2cc” refers to a coupler according to IEC 60318-5. “Ear simulator” refers to a coupler according to IEC 60318-4. Applied versions: IEC 60118-7:2005, IEC 60118-0:1994 and ANSI S3.22:2009.

FEATURE OVERVIEW	CPx	CP	N	NR	ITED	ITCPD	ITCD	ITC	CICP	CIC
<b>SIGNAL PROCESSING</b>										
ChannelFree™	●	●	●	●	●	●	●	●	●	●
Frequency Bandwidth	8 kHz	8 kHz	8 kHz	8 kHz	8 kHz	8 kHz	8 kHz	8 kHz	8 kHz	8 kHz
<b>LISTENING COMFORT</b>										
Adaptive Noise Reduction Plus	2 ctr	2 ctr	2 ctr	2 ctr	2 ctr	2 ctr	2 ctr	2 ctr	2 ctr	2 ctr
Adaptive Feedback Canceller Plus	●	●	●	●	●	●	●	●	●	●
Wind Noise Monitor	–	●	●	●	●	●	●	–	–	–
Soft Noise Management	2 ctr	2 ctr	2 ctr	2 ctr	2 ctr	2 ctr	2 ctr	2 ctr	2 ctr	2 ctr
<b>BINAURAL SYNCHRONIZATION</b>										
Volume Control, Program Change	●	●	●	●	●	●	●	–	–	–
<b>DIRECTIONALITY CONTROLS</b>										
Fixed Directional	–	●	●	●	●	●	●	–	–	–
Fixed Omni	●	●	●	●	●	●	●	●	●	●
Adaptive Directionality	–	●	●	●	●	●	●	–	–	–
<b>CONVENIENCE FEATURES</b>										
VC Clicks	●	●	●	●	●	●	●	●	–	–
Mute Via Push Button	●	●	●	●	●	●	●	●	●	●
Configurable Start-Up Delay	●	●	●	●	●	●	●	●	●	●
<b>INDIVIDUALIZATION</b>										
Program Options/Memories	10/4	10/4	8/4	9/4	9/4	9/4	9/4	7/4	6/4	6/4
Data Logging	●	●	●	●	●	●	●	●	●	●
Language Specific Targets	●	●	●	●	●	●	●	●	●	●
REMfit™	●	●	●	●	●	●	●	●	●	●
<b>WIRELESS / ACCESSORIES (OPTIONAL)</b>										
Remote Control (RC-P)	○	○	○	○	○	○	○	–	–	–
SoundGate 2 (Bluetooth®)	○	○	○	○	○	○	○	–	–	–
TV Adapter 2	○	○	○	○	○	○	○	–	–	–
Phone Adapter 2	○	○	○	○	○	○	○	–	–	–
FM/DAI Adapter	○	○	–	–	–	–	–	–	–	–

● standard ○ optional

**BTE AND CUSTOM INSTRUMENT COLORS**

All BTE colors are available for all four BTE styles.



All custom hearing instruments are available in the four colors shown below.





**ACCESSORIES (OPTIONAL)**

Product	Description	Part number
Remote Control (RC-P)	Discreet device for volume and program adjustment	160-02-350-00
SoundGate 2 (Bluetooth®)	Interface for wireless communication, remote control. With telecoil.	131231
Charger Cradle	SoundGate 2 charging accessory	130834
TV Adapter 2 (Bluetooth®)	Enables wireless reception of TV audio signals	127847
Phone Adapter 2 (Bluetooth®)	Enables wireless reception of landline phone calls	124396 (EU) 130976 (JP) 130977 (KR) 130978 (NZ) 130979 (US) 130980 (ZA) 130981 (AU) 130982 (BR) 130983 (CN) 131571 (RU)
DAI Adapter	For Carista CP/CPx BTE	399-50-521-00
FM Adapter	For Carista CP/CPx BTE	399-50-591-00



ACOUSTIC OPTIONS

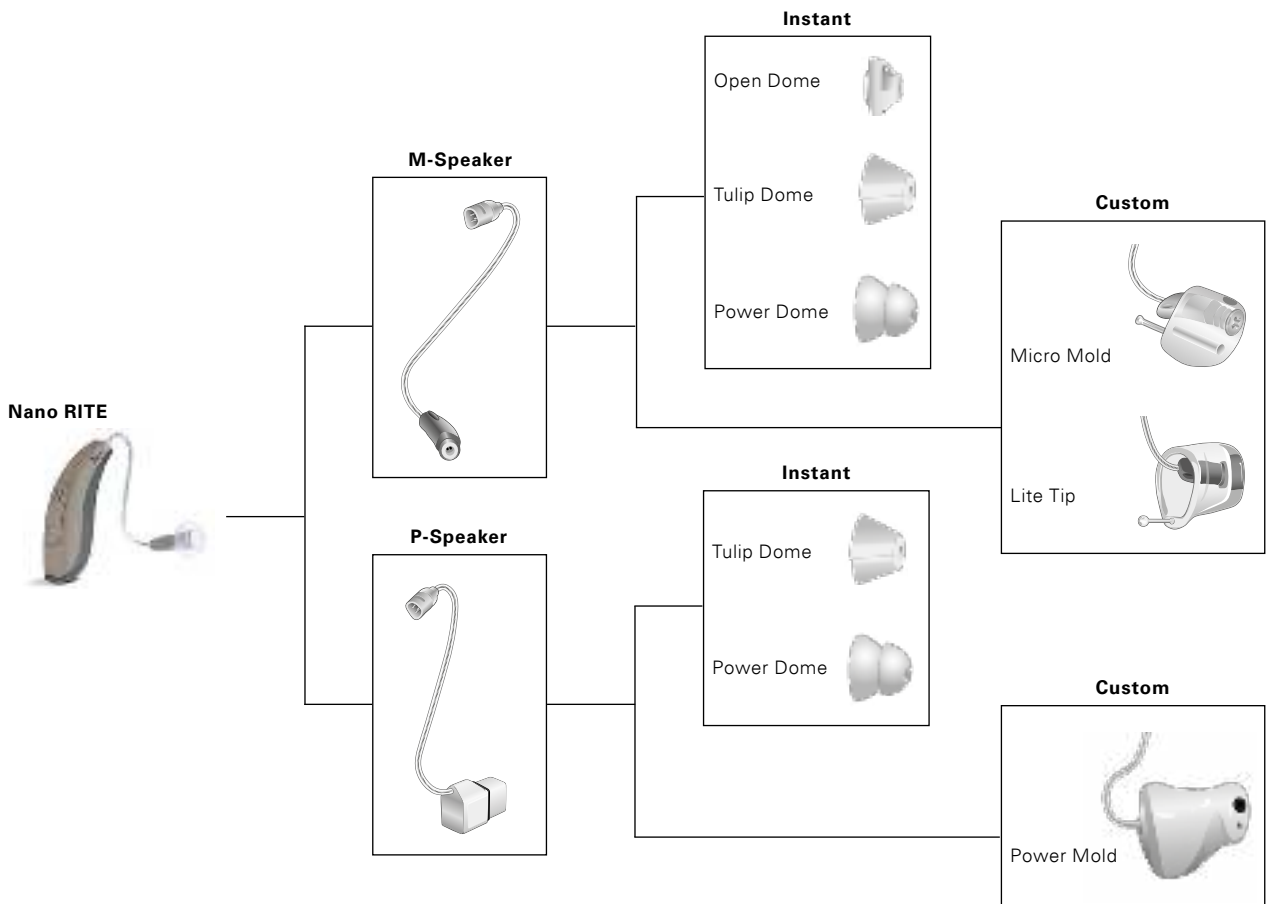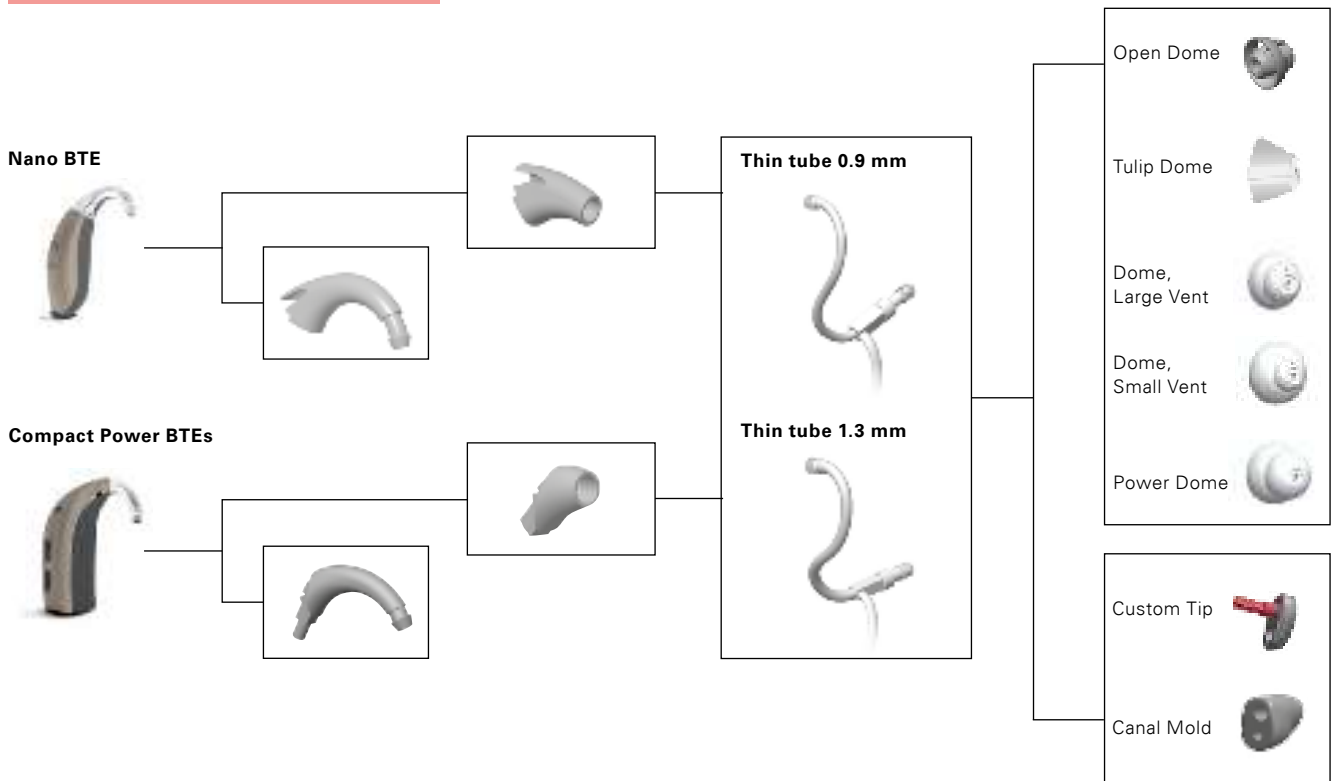

## FITTING KITS

Product	Description	Part number
Spira Flex Fitting Kit	Containing all Spira Flex parts. Upgraded with Power and vented domes.	890-80-060-00
Upgrade Kit for Spira Flex	Containing domes and parts to upgrade the Spira Flex Fitting Kit	122220
M-Speaker Kit	For Nano RITE	119979
P-Speaker Kit	For Nano RITE	119978



## PROGRAMMING EQUIPMENT

Carista 3 is programmed with Bernafon Oasis, version 19.0 or higher, a NOAH compatible MS-Windows® based PC-fitting software. NOAH with a HI-PRO, HI-PRO 2, NOAHlink, EXPRESSlink<sup>3</sup>, or nEARcom programming interface is required.

**Operating system**

Windows® 8, 32/64 bit, all editions  
 Windows® 7, 32/64 bit, all editions  
 Windows® Vista, 32/64 bit, all editions  
 Windows® XP SP3

**Noah**

Noah 4.3 (minimum for Windows® 8)  
 Noah 4  
 Noah 3.7 (minimum for Windows® 7)  
 Noah 3.6.1 (minimum for Windows® Vista)  
 Noah 3.5.2

Product	Description	Part number
Prog. cable, Nr. 2 New standard (HI-PRO)	Blue, left	384-20-033-00
Prog. cable, Nr. 2 New standard (HI-PRO)	Red, right	384-20-032-00
Prog. cable, Nr. 2 New standard (NOAHlink)	Blue, left	384-20-035-00
Prog. cable, Nr. 2 New standard (NOAHlink)	Red, right	384-20-034-00
Programming Adapter	For CPx/CP	399-50-640-00
FlexConnect Mini	For Carista custom instruments	117468

**Manufacturer**

**Switzerland**

Bernafon AG  
Morgenstrasse 131  
3018 Bern  
Phone +41 31 998 15 15  
Fax +41 31 998 15 90

**Australia**


Bernafon Australia Pty Ltd.  
629 Nudgee Road  
Nundah QLD 4012  
Freecall 1800 809 111  
Phone +61 7 3250 0300  
Fax +61 7 3250 0372

**New Zealand**

Bernafon New Zealand Ltd.  
Level 1, Building F  
27-29 William Pickering Drive  
Albany, Auckland 0632  
Toll Free 0800 44 22 57  
Phone +64 9 415 7917  
Fax +64 9 415 7916

**United Kingdom**

Bernafon UK  
Cadzow Industrial Estate  
Off Low Waters Road  
Hamilton  
ML3 7QE Scotland  
Phone +44 1698 285 968  
Fax +44 1698 421 456

**SWISS**   
Engineering

[www.bernafon.com](http://www.bernafon.com)

**bernafon**   
Your hearing • Our passion